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\*Gold MH et al. Dermatol Surg 2014;40: 825-831

## ADVANCED FORMULA SCAR TREATMENT

**KELO-COTE® Spray** is a lightweight, transparent, self-drying silicone gel for the treatment of scars. **KELO-COTE® Spray** rapidly dries to form a flexible, waterproof sheet which forms a protective barrier against chemical, physical and microbial invasion of the scar site while assisting with hydration. This creates an environment which allows the scar to mature through normalised collagen synthesis and helps improve the physiological and cosmetic appearance of the scar.

**KELO-COTE® Spray** has been shown to flatten, soften and smooth scars, relieve the itching, discomfort and pain of scars, as well as reduce the discolouration associated with scars.

Silicone gels have been shown to be effective in treating abnormal scarring in the form of hypertrophic scars and keloids. The 2014 International Clinical Recommendations on Scar Management supports the use of silicones as the preferred first line therapy for the treatment of linear hypertrophic scars, minor keloids and in the prevention of hypertrophic scars.\*

### Intended use:

**KELO-COTE® Spray** is for the treatment and prevention of hypertrophic and keloid scars resulting from burns, general surgical procedures and trauma wounds.

**KELO-COTE® Spray** can be used on joints, flexures, hairy areas and large area scars such as those caused by burns or major trauma. **KELO-COTE® Spray** is suitable for use on children over the age of 12 months and people with sensitive skin.

### Directions for use:

Shake can well before use. Ensure that the affected area is clean and dry. Remove cap. Hold can 15-20cm from surface to be treated. Spray along the length of the scar in a light, sweeping motion. **KELO-COTE® Spray** should be applied twice daily in a very thin, even coating on the affected area. Allow to air dry before covering with sun block or cosmetics. Replace cap after use.

**KELO-COTE® Spray** should dry in 4-5 minutes. If the **KELO-COTE® Spray** is pooling, dripping or running you have sprayed too much. Gently remove the excess and allow the drying process to continue.

Recommended minimum treatment time is 60-90 days. Larger and older scars may take longer and continued use is recommended if improvement is still seen after the initial 90 days.

### Caution:

For external use only. Do not use if you are hypersensitive to any of the ingredients in **KELO-COTE® Spray**. Do not use **KELO-COTE® Spray** near the eyes, nose or mouth. Avoid direct contact with mucous membranes, third degree burns and open wounds. **KELO-COTE® Spray** should not be used on dermatological conditions that disrupt the integrity of the skin. If irritation occurs, discontinue use and consult your physician. **KELO-COTE® Spray** may stain clothing if it is not completely dry.

### Danger. Extremely Flammable Aerosol.

Pressurized container: may burst if heated. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Do not pierce or burn, even after use.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition sources. Keep out of reach of children.

Store below 25°C. Do not use after the expiration (EXP) date printed on the can.

**Ingredients:** Butane, Pentane, Disiloxane, Dimethicone, Cyclopentasiloxane, Dimethicone Crosspolymer, Silica, Dimethiconol

**Contents:** 100ml

Lot number and expiry date are printed on the base of the carton.

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Advanced Bio-Technologies, Inc, 4830 West Kennedy Blvd, Suite 600, Tampa, Florida 33609, USA
Alliance Pharma S.r.l., Viale Restelli 5, 20124 Milano, Italien